

EN

D-125

Dental Suv

Used:

Disinfection of hard, non-porous surfaces such as counters, instrument trays, instruments tables, worktables, floors, furniture, beds, transportation cars, and similar items. For cleaning of hard surfaces prior to disinfection, cleaning of dental evacuation vacuum lines and for use in ultrasonic cleaning machines. For use in medical, sanitary and hospital areas, in social care institutions and sanatorium for long term sick. Kills broad spectrum of gram negative and gram positive bacteria's and envelope and non envelope viruses (including HIV-1, HHBV and HCV). Alkyl dimethyl benzyl ammonium chloride: 23,77 g/l. Alkyl dimethyl ethylbenzyl ammonium chloride: 23,77 g/l. <5% non-ionic surfactants, EDTA and salts thereof, cationic surfactants

Dilution preparation and use:

Add 1.6% or 80 ml of D-125 into the 5 Litres of water. (1:64). Remove gross filth or heavily soil. For heavily soiled area, a pre-cleaning step is required. Allow solution to contact surface such that all surfaces remain wet for 10 minutes. Let air dry. Prepare a fresh solution for each use.

Storage and Disposal:

Do not store on side. Avoid creasing or impacting of side walls. Liquid concentrate. Store in closed original container in a frost-free, cool, dark location. Shelf life of 12 months if unopened. If product freezes, thaw at room temperature and shake gently to remix components. Use locked storage in an area that will prevent cross-contamination of other biocidal products, fertilizer, food and feed. Store in locked area inaccessible to children. Do not contaminate water, food, or feed by storage or disposal. Do not discharge effluent containing this product into lakes, streams, ponds, estuaries, oceans, or other waters. Do not discharge effluent containing this product into sewer system without previously notifying the local sewage treatment plant authority. Do not reuse empty container.

Effectiveness:

Efficacy was demonstrated against common organisms in 5% organic soil load at 400 PPM hard water, unless otherwise noted on label. **The most important organisms, against which the product is effective*:** Human Hepatitis B Virus, Hepatitis C Virus, HIV - 1 (AIDS Virus), Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella choleraesuis, Enterococcus hirae, Trichophyton mentagrophytes (Pathogenic Fungi), Escherichia coli, Herpes Simplex type 2 virus, Polio type 1 (Cat Strain) 30 minute contact time, Influenza A/Victoria (H3N2) Virus, *In 98% soil load and 791 ppm Hard Water demonstrated efficacy within 10 minutes against Staphylococcus aureus and Salmonella chleraesuis.

Product-Types (PT-No.): 02



Irritant

Irritating to eyes. Keep out of the reach of children. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

This material and its container must be disposed of in a safe way. If swallowed, seek medical advice immediately and show this container or label.

Microgen-Europe B.V.,
Rokin 55, NL-1012 KK Amsterdam.
Tel.: +31 205 214 - 777,
Fax: +31 205 214 - 888

Batch number:

0,5 l



MICROGEN-EUROPE BV

Disinfectant Products, Redefining Clean




PANTONE
259c



PANTONE
2602c



PANTONE
151c

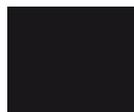

PANTONE
Black c