6"

PDI®

The Professionals' Choice for Infection Control

Sani-Cloth HB

Disposable Disinfection Wipes for Medical Devices

Providing Bactericidal (EN1040, EN 1276), Fungicidal (EN 1275), & Virucidal disinfection.

- F** Lingettes de désinfection à usage unique pour dispositifs médicaux
- D** Einmaldesinfektionstücher für medizinische Geräte
- NL** Disposable desinfecterende doekjes voor medische hulpmiddelen
- I** Salviette disinfettanti monouso per dispositivi medicali
- S** Toallitas desinfectantes desechables para dispositivos médicos

For Professional and Hospital Use

Sani system

160 Wipes, Lingettes, Tücher, Doekjes, Salviette, Toallitas.

Alcohol Free

CE 0459

3.75"

NP# 834 – Effective 1/15/04
add 1/8" bleed if needed

6"

3.75"

GB Sani-Cloth HB is contraindicated for use as a terminal sterilant/high level disinfectant on any surface or instrument that is a) Introduced into the body or b) may come into contact with the bloodstream, including broken skin or mucous membranes. Sani-Cloth HB can be used to pre-clean or decontaminate medical devices prior to sterilisation or high level disinfection. Caution: For external use only. Keep product closed and stored upright in a cool dry place.

Special Instructions for cleaning and decontamination against HIV1 and HBV of surfaces/objects soiled with blood or body fluids. Personal Protection: Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable protective gloves, gowns, masks, or eye coverings. Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surface and objects before application of the disinfectant. Contact Time: For viruses, leave surface wet for 1 minute (10 minutes required for HBV and HIV-1). For mould and mildew control, leave surface wet for 5 minutes.

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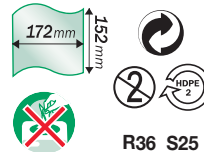
**Composition/ Composition/
Inhalt/ Ingredienti/ Contiene:**
Surface wipe saturated with
Alkyl dimethyl benzyl ammonium
chloride 0.07%. Alkyl dimethyl
ethylbenzyl ammonium chloride
0.07%. Inert ingredients 99.86%
Licensed by Microgen Inc.

REF: Q59072EX

PDI Aber Park, Flint, UK.
Tel. +44(0)1352 736700
Email. pdi@nice-pak.co.uk

PMS 485c

PMS 347c

4 Process

NP# 834 – Effective 1/15/04
add 1/8" bleed if needed