6"

PDI®

The Professionals' Choice for Infection Control

Sani-Cloth HB

Disposable Disinfection Wipes for Medical Devices

Providing Bactericidal (EN1040, EN 1276), Fungicidal (EN 1275), & Virucidal disinfection.

- F** Lingettes de désinfection à usage unique pour dispositifs médicaux
- D** Einmaldesinfektionstücher für medizinische Geräte
- I** Salviette disinfettanti monouso per dispositivi medicali
- S** Toallitas desinfectantes desechables para dispositivos médicos
- P** Toalhetes para desinfeção descartáveis, próprios para dispositivos médicos

- ✓ Virucidal
- ✓ Fungicidal
- ✓ Bactericidal

Sani system

160 Wipes, Lingettes, Tücher, Salviette, Toallitas, Toalhetes

For Professional and Hospital Use

Alcohol Free

Reference 1103055

3.75"

NP# 834 – Effective 1/15/04
add 1/8" bleed if needed

6"

3.75"

GB Sani-Cloth HB is contraindicated for use as a terminal sterilant/high level disinfectant on any surface or instrument that is a) introduced into the body or b) may come into contact with the bloodstream, including broken skin or mucous membranes. Sani-Cloth HB can be used to pre-clean or decontaminate medical devices prior to sterilisation or high level disinfection. Caution: For external use only. Keep product closed and stored upright in a cool dry place.

Special Instructions for cleaning and decontamination against HIV1 and HBV of surfaces/objects soiled with blood or body fluids. Personal Protection: Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable protective gloves, gowns, masks, or eye coverings. Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surface and objects before application of the disinfectant. Contact Time: For viruses, leave surface wet for 1 minute (10 minutes required for HBV and HIV-1). For mould and mildew control, leave surface wet for 5 minutes.

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