



Disinfectant Cleaner Virucide and Fungicide

D - 125

Dental SUV



Uses:

Disinfection of hard, non-porous surfaces such as counters, instrument trays, instruments tables, worktables, floors, furniture, beds, transportation cars, and similar items. For cleaning of hard surfaces prior to disinfection, cleaning of dental evacuation vacuum lines and for use in ultrasonic cleaning machines. For use in medical, sanitary and hospital areas, in social care institutions and sanatorium for long term sick. Kills broad spectrum of gram negative and gram positive bacteria's and envelope and non envelope viruses (including HIV-1, HHBV and HCV)

Dilution preparation:

Add 1.6% or 80 ml of D-125 into the 5 Litres of water. (1:60). Allow solution to contact surface for at least 10 minutes and then wipe or let air dry.

Effectiveness:

Efficacy was demonstrated against common organisms in 5% organic soil load at 400 PPM hard water, unless otherwise noted on label.



Microgen-Europe BV

www.microgen-europe.com

Contents 500 ml

Safety information

R-36 Irritating to eyes

S-02 Keep out of the reach of children

S-26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice

S-35 This material and its container must be disposed of in a safe way.

S-46 If swallowed, seek medical advice immediately and show this container or label.



D 125 contains

less than 5% nonionic surfactant, EDTA, cationic surfactant

The most important organisms, against which the product is effective*:

Human Hepatitis B Virus

Hepatitis C Virus

HIV - 1 (AIDS Virus)

Pseudomonas aeruginosa

Staphylococcus aureus

Salmonella choleraesuis

Enterococcus hirae

Trichophyton mentagrophytes (Pathogenic Fungi)

Escherichia coli

Herpes Simplex type 2 virus

Polio type 1 (Cat Strain) 30 minute contact time

Influenza A/Victoria (H3N2) Virus

*In 98% soil load and 791 ppm Hard Water demonstrated efficacy within 10 minutes against Staphylococcus aureus and Salmonella choleraesuis.

Batch number: 123456 R

Store in a frost-free, cool, dark location. Shelf life of 12 months if unopened.

Sold by:

Microgen - Europe B.V., Rokin 55, 1012 KK Amsterdam, The Netherlands

Telephone +31(0)20 521 • 4777 Fax: +31(0)20 521 4888

Produced by: Micorgen Chemie BV, Rokin 55, 1012 KK Amsterdam, The Netherlands